



Premium Hospital Beds

LUNA G



LONG-TERM CARE

Electric Control



Inclusive Healthcare

With simple electrical functions, the Luna G geriatric bed is inserted as an economical alternative in order to create affordable quality health.

The robustness and reliability of the entire system provide the necessary peace of mind for effective care.

Robust Architecture

Lifting system by trapezoidal arms electrically articulated by linear motor.

Lifting stroke 420-820 mm and maximum patient load of 220 Kg.

¹ Handset Control

Portable control powered by an extendable helical cable.

Allows the following positions:


- Back
- Legs
- Auto Contour
- Height

² Metallic Structure

The metallic structure with an electrostatic epoxy paint finish with prior anti-corrosion treatment means that the product is suitable for hospital cleaning and disinfection with alcohol-based detergents and damp cloths with lukewarm water

Auto Contour

Automatically performs, by the activation of a single button, the position of Auto Contour, raising the leg and back sections in sync. Ensures a correct patient positioning when raising his back.

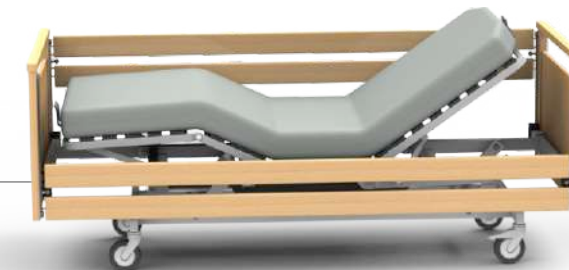

Bed Height

Adjusts the height of the bed platform through electrically articulated trapezoidal arms. With a 420 mm lifting stroke, it allows positioning the patient for ergonomic medical interventions for the healthcare professional.



Shock

It performs automatically, by the activation of a single button, the Shock by raising the leg and foot sections in sync. Used to reverse hypoxic episodes in vital organs.



Preventive Locking

Both guard rail options are fitted with safety handles. These are in a place which is difficult for the patient to access, but are very ergonomic for healthcare professionals.



Care Accessories

Available next to the head panel are two removable accessories fittings which may facilitate the procedures in various therapeutic frameworks.

Safe Bed Frame


The Luna G bed boards are equipped with ergonomic polyamide bulkheads to secure the mattress during the movements of articulation.


Mattress Platform

Articulated bed frame with 4 sections, made up of flexible slats in steamed beech wood with PVC terminals.



Anti-Bedsore Mattresses

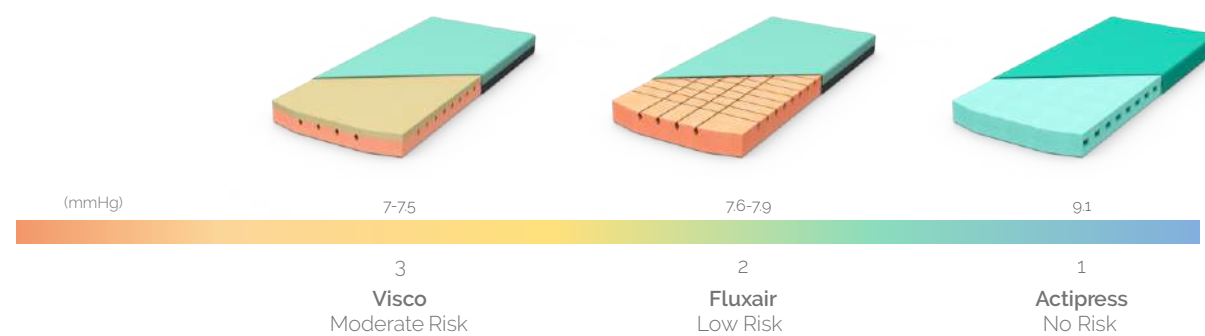

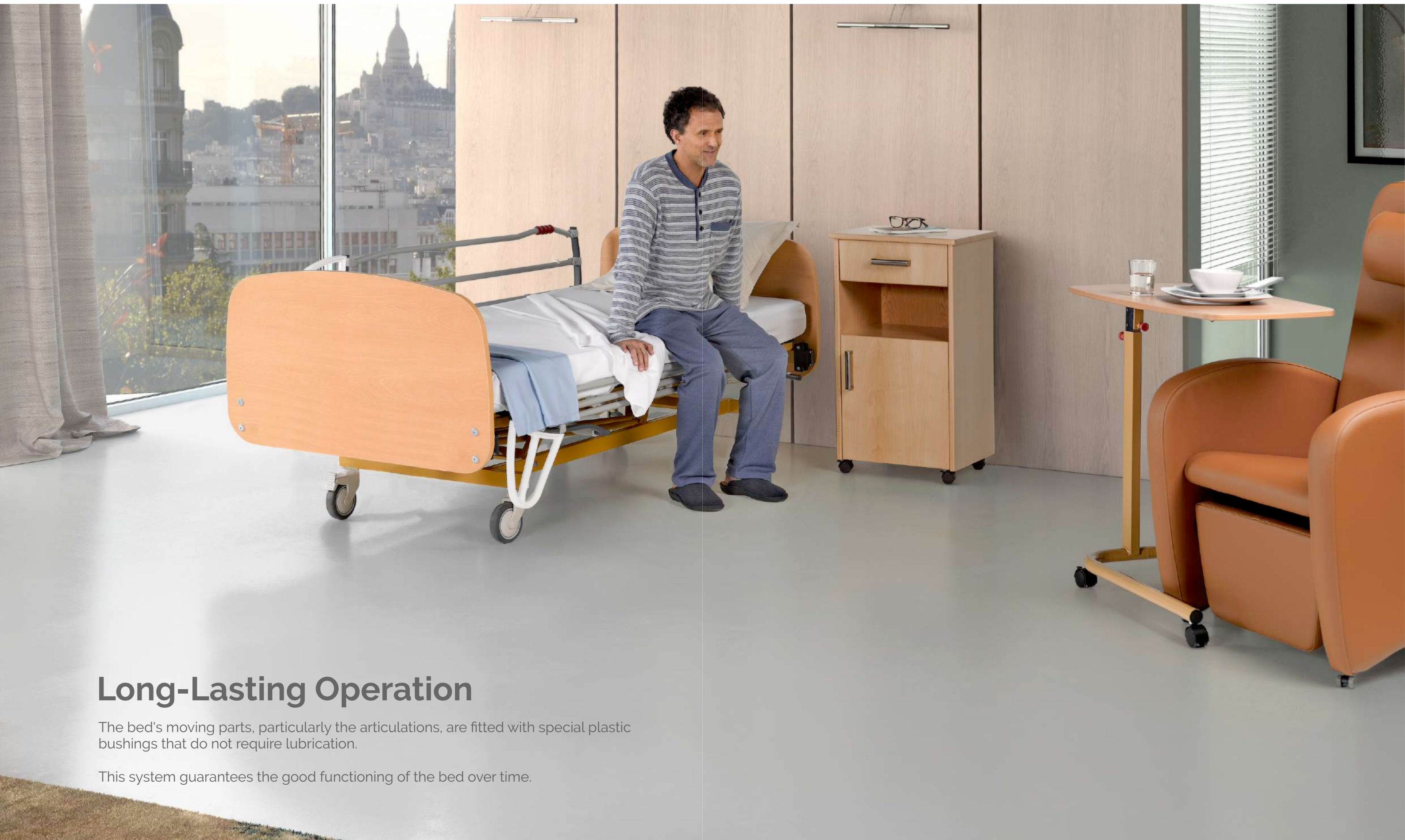

Long-Lasting Operation

The bed's moving parts, particularly the articulations, are fitted with special plastic bushings that do not require lubrication.

This system guarantees the good functioning of the bed over time.

SAFETY

Sides Rails Option



Telescopic Side Rails

Both side rails options comply with EN 60601-2-52 standard. They prevent and avoid the risk of injuries caused by accidental falls through high side coverage.

Option of full-length wooden side rails, foldable under the mattress.



Tilting Side Rails

Option of full-length side rails in epoxy-painted steel tube and plastic ear extension.

Folding, with safety lock and ergonomic handle for movement.

TRANSPORT

Caster Options



1

Faired Casters

Bed option with 4 casters faired with ABS Ø125 mm. Rotating effect and fitted with ball bearings.

Choice of 2 or 3 local brake casters.



2

Metallic Casters

Optional bed with 4 metallic casters Ø100 mm. Rotating effect and fitted with ball bearings.

2 casters with local brake

TECHNICAL FEATURES

Model 954R

BED ACCESSORIES

		Standard	Optional		
					IV Pole
Structure	Robust metal frame epoxy painted	●			Serum support Stainless steel column Adjustable rod with 4 hooks 910.104i
	Mobile components zinc plated or/and stainless steel	●			Serum support Epoxy painted column Adjustable rod with 4 hooks 910.104
Mattress Platform	4 sections bed frame (3 articulated + 1 fixed).	●			
	Slats in steamed beech wood, PVC terminals	●			
	Flexible mattress shoulders	●			
					Lifting Trapezee
Base	Open design base to prevent dirt build-up	●			Patient lifting Stainless steel column Trapezee with adjustable handle 910.200
	Ø125mm ABS casters with local brake		●		
	Ø100mm zinc steel casters with local brake		●		Patient lifting Stainless steel column Trapezee with auto-adjustable handle 910.201
Side Rails	Full-length, in natural beech wood, biphasic release buttons		●		
	Full-length, metallic with epoxy coating, soft drop handle		●		
					Crutches Holder
Control	Handset control with extendable helical cable	●			Support for 2 crutches Rubber fitting Epoxy painted structure Removable and adjustable 910.570
Electric Setup	Low voltage motors DC24V (3 actuators)	●			

LUNA G

* Ø100 mm zinc steel casters

Dimensions +/- 5%

Recommended mattress dimensions	1970 x 860 x 120-150 mm
External dimensions (wooden rails)	2110 x 1040 mm*
External dimensions (metal rails)	2100 x 1000 mm*
Min/Max height	420 - 820 mm*

Joints +/- 5%

Maximum back incline	70°
Maximum leg incline	35°
Maximum Auto Contour incline	82°

Capacities +/- 5%


Safety load (SWL)	220 Kg
Power supply	220-230 VAC, 50 Hz
Control unit power	140 VA



IMO - Indústrias Metalúrgicas, S.A.

Headquarters

Rua Dr. Aurélio Teixeira de Sousa, 575
4650-312 Rande Portugal
T. +351 255 340 220
sede@imo.com.pt

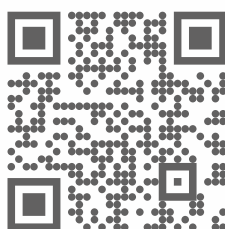

www.imo.com.pt